## OrthoPro® Stabilizer Knee Brace

**TREATMENT RATIONALE:** Facilitate maintenance or rehabilitation of the knee. The ROM hinge can be set to control both flexion and extension motion in the brace. Ideal for providing protected ROM exercise in the brace post surgically or to limit motion to prevent further injury of the joint during rehabilitation. Can also be used at a static setting to protect and support the joint.

**FUNCTIONAL OBJECTIVE:**

Increase walking and other activities while supporting the knee joint to prevent further knee injury thereby reducing the inflammation and knee discomfort that may be associated with unsupported use.

**ORTHOTIC APPLICATION AND FITTING:**

1. The orthotic fitter should pre-set the ROM hinges on both uprights per physician's instructions.
2. Apply the neoprene knee wrap oriented so that the top of the patella opening is aligned just above the top of the knee cap.
3. Center the brace so that the metal uprights are in the same alignment (parallel) on either side of the knee with the hinge center at mid-patella.
4. Close the thigh cuff flap snugly by folding the non-hook closure flap over the thigh with the top edge of the brace aligned. Gently press down over the hook closure to ensure that the hook is securely attached.
5. Close the shin flap closure in the same manner as the thigh cuff closure. Gently press down over the hook closure to ensure that the hook is securely attached.
6. Bring the tab end of the upper strap through the plastic "D" ring and secure the hook end back onto the loop side of the strap. The strap should be comfortably snug providing directional support.
7. Bring the tab end of the lower strap through the plastic "D" ring and secure the hook end back onto the loop side of the strap. The strap should be comfortably snug providing directional support.

**\*\* NOTE: The brace comes with the medial and lateral (upper and lower) straps situated so that one strap pulls in a medial direction and the other in a lateral direction. These straps can be detached on the posterior side of the brace and repositioned to provide the appropriate realigning force. (medial/lateral).**

8. Adjust the straps to comfort during brace wear.
9. Always check for redness or skin irritation when removing the knee brace. Redness should resolve itself within several minutes after the brace has been removed. Do not use the brace and contact your orthotic clinician immediately if redness persists or there is pain and discomfort with use.

**LAUNDRY INSTRUCTIONS:**


**KNEE ORTHOSIS:**

- Remove the rigid side bars from the brace.
- Fasten all hook and loop closures.
- Hand or machine wash, gentle cycle with mild detergent. **DO NOT USE COMMERCIAL WASHERS OR HOT WATER.**
- No bleach or fabric softener.
- Air dry.

**HINGES:**

- Clean with mild detergent or antimicrobial wipe
- Rinse with clean water
- Dry with soft cloth

**WARNING: The product should be fit by trained personnel. The product is designed for single patient use only in order to avoid cross contamination. Any substitution or removal of the product's parts voids the manufacturer's warranty. Ongoing Care Solutions, Inc. will assume no liability if the above instructions are not followed.**

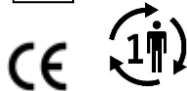Authorized Representative  
MDSS GmbH, Schiffgraben 41  
30175 Hanover, Germany

Ongoing Care Solutions, Inc  
11721 US HWY 19 N  
Clearwater, FL 33764  
800-375-0207 (P) 727-525-1424(F)  
[www.ongoingcare.com](http://www.ongoingcare.com)

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